

# LockeT

Intuitive Suture Retention Device

The LockeT device offers a cost-effective solution for site closure with suture retention, ensuring efficient hemostasis while prioritizing patient comfort. Compact and user friendly, LockeT simplifies the closure process, making it easy to monitor, adjust and remove as needed.



Simple design,  
easy to use



Patient  
Comfort



Cost Effective



Suitable for a wide  
range of catheter  
sizes up to 27F




Quick  
Hemostasis




Transparent  
design for easy  
monitoring

# LockeT


Intuitive Suture Retention Device

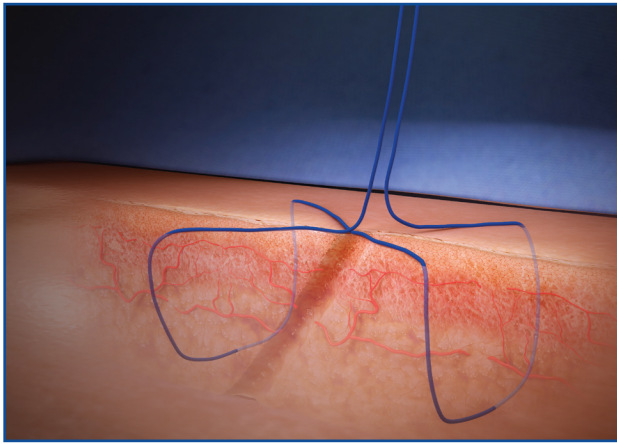

**Step 1: Create a standard figure of eight and remove sheaths.**

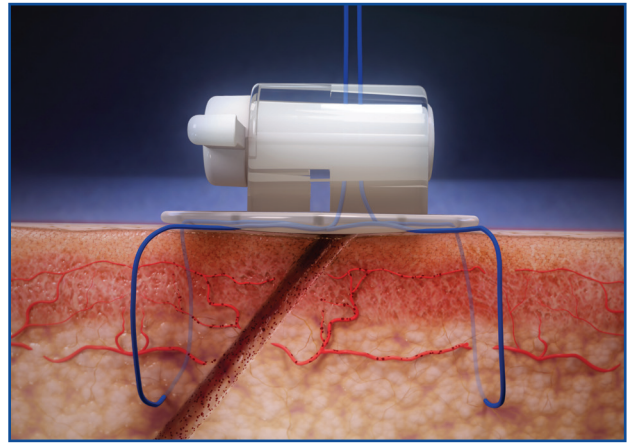

**Step 2: Tighten sutures, cleanse site and place sutures within LockeT.**

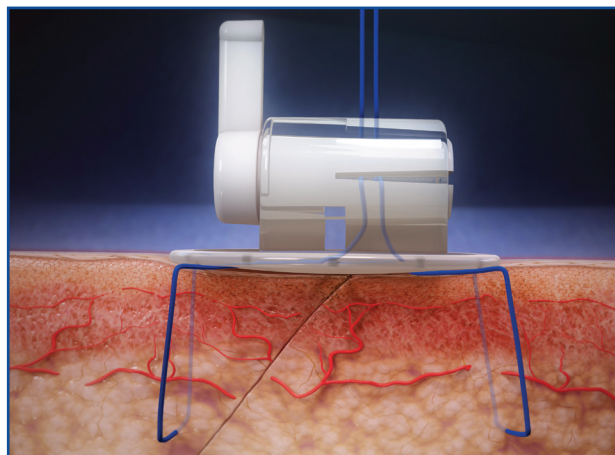

**Step 3: Rotate the handle 90 degrees.**

## ORDERING INFORMATION

MODEL NUMBER	PRODUCT DESCRIPTION	QUANTITY
9301	Suture Retention Device	1 Box (10 devices/box)

LockeT is a Class 1 sterile product registered with the FDA under Device ID D496327

Refer to IFU for complete Indications for Use, contraindications, warnings, and precautions.



PN 82-9300-001 Rev02w  
©2024 Catheter Precision, Inc. All Rights Reserved

**Catheter  
Precision**

1670 Highway 160 West | Suite 205 | Fort Mill, SC 29708 USA  
T: (+1) 973-691-2000 | F: 973-691-7573  
[www.CatheterPrecision.com](http://www.CatheterPrecision.com)