


Vivo | POSITIONING PATCHES

REMOVE BY
PEELING OFF

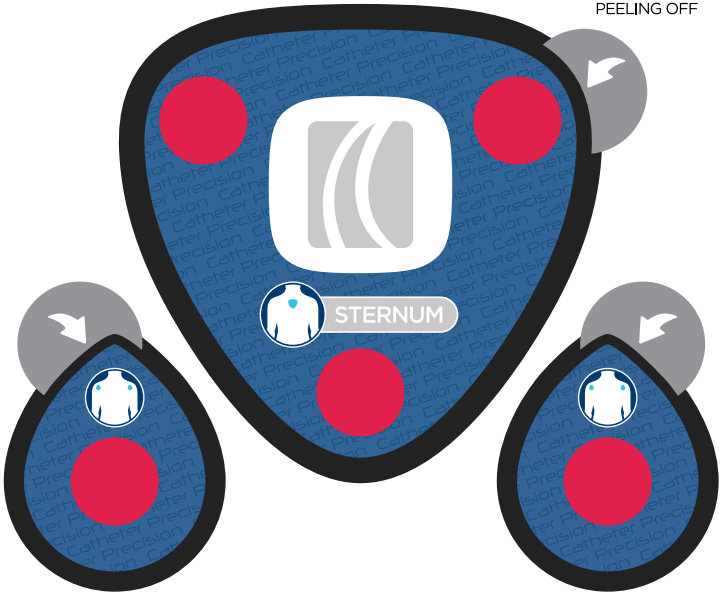

These positioning patches are single use only.



Enter the serial number for the patches into the VIVO application. The 3D image cannot be taken unless the one time use serial number is entered.



Catheter Precision Inc.

1670 Highway 160 West, Suite 205
Fort Mill, SC 29708, USA
Telephone: +1 (973) 691-2000
www.catheterprecision.com



USE BY:



YYYY-MM-DD

SN

###-###-###

Rx only

Federal (USA) law restricts this device to sale by or on the order of a licensed physician.

POSITIONING PATCH APPLICATION INSTRUCTIONS

ATTACHING THE POSITIONING PATCHES


Application sites must be clean, dry and free of any body lotion to provide for optimal patch adhesion to skin. Follow standard practice for attaching/removing ECG electrodes.

STERNUM POSITIONING PATCH PLACEMENT

The sternum patch should be placed with the top of the patch in the center (green dotted vertical line) of the sternum.

SHOULDER POSITIONING PATCH PLACEMENT

Place the Right and Left Shoulder patch so that they define the outer boundary of the body laterally (blue dashed vertical lines), and are parallel at the shoulder height (red solid line).

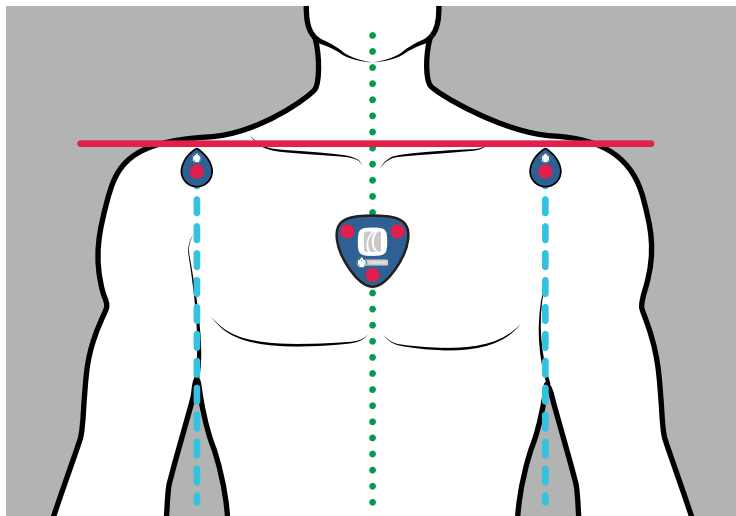

CAUTION: if the patches are not placed in the correct anatomical location, the VIVO results may be incorrect.

Questions or Comments?

Contact us at CustomerService@CatheterPrecision.com